

# CARE & SAFETY INSTRUCTIONS



# Repose

Furniture

## **Thank you for the purchase of a Repose recliner chair**

Repose would like to ensure you receive optimum life from your chair. To help you achieve this, please read this user manual so that with the right care, attention and use, your chair will stay in excellent condition and most importantly be used safely for the protection of you and those around you.

**COMFORT  
WITHOUT COMPROMISE**

# IMPORTANT SAFETY INSTRUCTIONS

Repose are confident that you will be pleased with your purchase and hope that the information included in this brochure will provide you with all the necessary information to aid easy use and maintain the life of your purchase. Prior to using the chair it is vitally important you read this information to ensure not only your own safety but that of others in the vicinity and to avoid damage to your furniture.

## INSTALLATION

- It is the users responsibility to ensure that they are fully aware of all the information in the leaflet, however, if for any reason you are unsure of anything, please contact your local stockist for further information.

**Prior to installation and use: Please ensure your chair will not be hindered by any object as this will affect the performance and damage the action.**

- Position the chair free from any obstruction and trailing wires
- Position the back onto the brackets (using a slight downward pressure) and refit velcro back. Depending on the mechanism you may find a small lever attached to the bracket, which should be pulled towards you to lock the back into position.
- The chair should be positioned with enough room for the chair to fully recline.
- If you have a power chair, for safety reasons check the extension cable (that will be located at the rear) has not been damaged in transportation. On satisfaction connect to your transformer.
- Once your transformer has been connected, plug the unit into the power source and switch all power on ready for use.  
NB: This product is not unsafe when power source is left on.
- If you need to remove the back, release the locking bars or clips (model dependant) and pull the back upwards.
- When moving or handling the chair DO NOT use the backrest, use the base only. Using the backrest may cause damage to the internal framework of the chair.


# HAND INSTRUCTIONS

PLEASE DO NOT ALLOW CHILDREN TO PLAY WITH THE HAND CONTROL.

Handsets come in varying designs depending on the type of mechanism inside the chair. We have included the following diagrams and instructions to aid easy use of your rise and recliner.


## Single Motor Handset Dewert/Okin

- Button 1 – back up, footrest down, chair rises
- Button 2 – footrest up, back down, chair lowers
- Button 3 – reset (puts chair to sitting position)

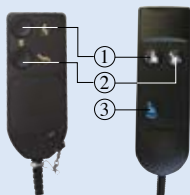


## Dual Motor Handset Dewert/Okin

- Button 1 – back up
- Button 2 – lifts chair, footrest in
- Button 3 – back down
- Button 4 – chair lowers, footrest out
- Button 5 – reset (puts chair to sitting position)

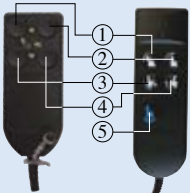


## Dual Motor Handset (6 button)

- Button 1 – back down
- Button 2 – back up
- Button 3 – chair lowers, footrest out
- Button 4 – chair lifts, footrest in
- Button 5 – chair lowers, footrest out
- Button 6 – chair rise, footrest in

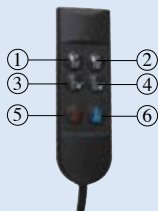

## Dual Motor Handset Stawert

- Button 1 – chair lowers, footrest out
- Button 2 – chair lifts, footrest in
- Button 3 – back down
- Button 4 – back up

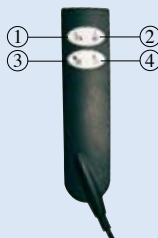

## Heat and Massage Handset

Instructions will be supplied with the handset.

# IF YOU EXPERIENCE ANY TECHNICAL DIFFICULTIES YOU SHOULD:

- Always disconnect the recliner from the power before checking any part of the chair
- Check the wall socket is in working order by trying another appliance
- Check the connection from the chair to the transformer is connected
- Check to see if any of the cables are damaged, check fuse and batteries (where applicable)
- Do not position your chair too close to obstacles that may cause damage, the zero wall units can be placed 3"-4" (8-10cm) from the wall without causing a collision, but other recliners must be positioned well away from obstacles
- Store your hand controller in either the pocket or on the loop provided. This will ensure no damage by it accidentally falling onto the floor or person(s) sitting on the unit accidentally
- Do not allow children to play on or with the chair
- Do not move the chair whilst a person is sitting in it

**If you are in doubt about any of the above, please contact your supplier direct.**

## UPHOLSTERY CARE

- Small dust and dirt particles can increase wear and tear of the fabric and dull the surface appearance, therefore we recommend that you brush your furniture using a soft brush or lightly vacuum. **Never use abrasive tools or cleaning fluids.**
- Repose Furniture cannot be held responsible for any damage caused by cleaning and so we strongly advise you to ask your stockist for information prior to cleaning if you are unsure.
- Your upholstery should not be exposed to direct sunlight for long periods of time as this will cause your upholstery to fade. If your furniture is positioned within prolonged light, ensure your curtains and blinds are drawn for protection.
- With certain fabrics such as Draylons and Chenilles pile flattening may occur, which is not a problem but an inherent characteristic. This will appear as different shades and may give the impression of the cover fading or creasing, which is caused by the pile of the fabric being moved in different directions. If this does occur, we recommend gently brushing in the opposite direction or vacuuming and leaving it to settle for a while. In most cases this will bring the pile to its original position, however, you will experience some marking afterwards, which is unavoidable due to the nature of the fabric.
- Ensure your furniture is protected from sharp objects or buckles as these may cause the fabric to snag. If this does occur do not pull the thread, carefully trim the thread with scissors.
- Avoid allowing pets onto your furniture as their claws may cause damage to the fabric. Also, with all animals there will be a degree of oil in their coats so please remember by allowing them on your furniture the oils may transfer to your fabric. This will cause your fabric to prematurely wear. Arm caps and head covers can be obtained to help the most common wear areas and these are available from your local stockist.
- The arms are designed for resting your arms only, therefore, do not sit on them as this will cause structural damage to the product.

# WARRANTY

Your Repose chair comes with a 2 year warranty, Repose will repair or replace at our option to the original purchaser, any of the following parts found upon examination to be defective in material and/or workmanship.

- Wood frame
- Motor
- Hand control
- Scissor mechanism

**Note:** warranty is for parts only and will only include transportation to and from the original delivery point. The warranty exclusions are: fabric; damage resulting from misuse; unauthorised repairs; abuse; negligence; accident(s) or alteration and modifications of the product; damage caused by common carrier abuse or mishandling; any special, indirect, incidental and/or consequential damages.

**Never attempt to open any electrical components or dismantle the chair in any way as this will deem your warranty invalid. Further assistance from your stockist will ensure the chair is repaired in a safe manner and so prevent injury to you or others.**

**For warranty claims Repose Furniture must be provided with the original purchase invoice number to validate warranty.**



Recommended stockist: