

Tandem User Instructions



Issued 1st September 2010



Tandem Buggy

- We designed the tandem buggy to enable families who have a disabled child and another younger child to enjoy the freedom they deserve. The system is not designed for two children who both have special needs as we are not able to provide postural support in the second seat.
- In order to create a buggy that can take two children, and can still be pushed easily and navigate around shops and other urban terrain, there had to be a element of compromise. One of these is balance. It is important that the front seat is always occupied when using the rear seat.
- **RULE NO.1 – Do not put a child in the rear seat if the front seat is not occupied**
- **RULE NO.2 – Do not take the child out of the front seat if the rear seat is still occupied**
- This is a very important thing to explain to other users and carers, and will ensure that both occupants remain safe.
- Care should also be taken when adjusting the positions of either seat to ensure that the other seat still has enough space. They should be positioned as close to each other as possible to keep the weight over the wheelbase as much as possible.

Safety notes – please do read these!

- Please read this user guide fully before using your stroller.
- **The Maximum Weight Capacity for the Main Front Seat is 35kg. The Maximum weight for the rear baby seat is 9kg and the rear sibling seat has a maximum weight capacity of 20kg.**
- Expert medical advice must be sought before using your stroller. If you have any doubt about this speak to your Occupational Therapist.
- Before using the stroller it is essential to check that the Frame Catch is locked, and that the Tilt-in-space mechanism and the seat recline are tightened.
- Each attendant should be familiar with the operation of the stroller. If there is any doubt they should read this User Guide.
- Before setting off, ensure that the occupant is comfortable, and that clothes and rugs are tucked in, so that they do not catch on the wheels.
- Always apply the parking brake when the stroller is stationary and when getting in and out of the stroller.
- Never tip the stroller forward when going down a kerb.
- If the stroller is going to be used for transport in a vehicle you should carefully read the Transport Section of this User Guide.
- The attendant must take precautions against fire risk. In buildings, check the locations of fire exits and make sure the emergency procedures are understood.
- Do not let children play with, or hang on the stroller.
- Do not use your stroller in a manner for which it is not designed.
- Ensure that children are clear of moving parts before making adjustments.
- Patron Bohemia, Activate, and your dealer, cannot accept responsibility for the use of accessories other than those supplied by them.
- Full concentration must be maintained during travel to avoid collisions or other dangerous situations.
- Make sure that the stroller brakes are applied while adjusting the recline or the tilt of the seat. This will leave both hands free to operate these features, avoiding sudden movements which will cause discomfort to the occupant.
- The seat base attachment must not be released while the stroller is occupied. Make sure that the seat is empty before removing the seat or changing its direction of travel.
- Do not overload the stroller. Other children should not ride on the back of the stroller, or loads attached to the frame other than on the approved accessories.
- Ensure that the occupant has the appropriate harness attached and adjusted to prevent them from falling out of the stroller while travelling. The attendant should know how to fit, adjust, and release these features.
- Do not stand on the footrest. The footrest should be folded whilst loading.
- Never leave the child unattended in the stroller, even for the shortest time.
- At all times when operating the stroller, be aware of the danger of injury to yourself, the occupant or any other person, especially when going through doorways.
- Patron Bohemia, Activate, and your dealer, cannot accept responsibility for any loss, damage or injury resulting from the incorrect operation of your stroller.
- The Tandem buggy is expected to last 3-4 years with continual daily use and regular maintenance.

Operating the Tilt Function

- The angle of the seat unit is altered by releasing the tilt lever indicated on our main picture.
- The backrest of the seat should be held with one hand while releasing the lever with the other hand. Re-position the seat angle and let go of the lever, checking that the seat is secure in the new position.



Always check that the child in the rear seat has enough space when tilting the seat

Removing and re-fitting the front seat unit

- The Seat unit should be removed to allow the frame to fold.
- The seat is secured by two levers under the seat unit at the rear as indicated in our first picture (you are looking down from above, with only a seat frame mounted on the chassis)



- These levers should be opened by pulling the pop-pins out and swinging the levers backwards as shown
- The seat unit then tips forward and is lifted off the interface.
- Make sure that the Seat carriage is placed in a level position for folding



Adjusting the push handle height/ folding the chassis

- The height of the pushing handle is adjustable, and this also enables the handle to be folded away for storage of the complete buggy – or when folding the frame
- The lever on the adjuster should be opened, which allows the gears to disengage. The handle can then be moved to the required position and the lever closed to secure the adjuster.



• Adjuster Closed



• Adjuster Open

Folding the Chassis

- The chassis is folded very simply by pressing the release handles together as shown
- Make sure that you have supported the handle frame, otherwise the chassis will fold on top of you!



- You will find that the chassis fits into most car boots, as it is short enough and shallow enough once folded.
- **Helpful Tip!** – lock the direction on the front castors by lifting and turning the pin and locating it into the castor housing. This will hold the castors in line while it is folded, making it easier to stow away.



Using the Brakes

- The brakes are fitted on either side of the Tandem Buggy, and both brakes should be applied while the buggy is being loaded or unloaded.
- Always make sure that the brakes are gripping the wheels sufficiently – especially when parking the buggy on slopes.




- Brake off



- Brake applied

Fitting the rear seats

- The Tandem buggy is designed to receive both a baby car seat that is suitable from birth to 9 months old, and a sibling seat with takes a child from 9 months up till 3-4 years old.
- The Carrying Frame for the Baby Car Seat and the Sibling seat fit into the receivers on the rear of the Tandem as shown
- The handwheels are tightened to secure the frame in position
- The backrest angle on the sibling seat is adjusted by pressing the button on both side of the seat
- **Please note – the rear seat must be removed when the buggy is used in occupied transport**

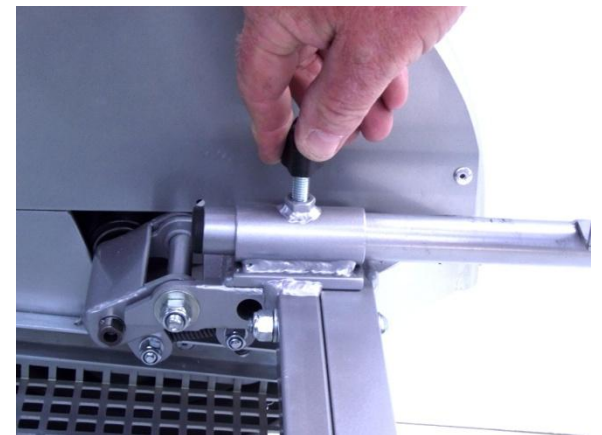

Adjusting the Recline Angle of the Seat unit

- You should be aware of the way this movement could affect the user of the rear seat.
- To release the position undo the lever handle as show, then pull on the pop-pin while supporting the weight of the backrest. Reposition and release the pop-pin to locate into the closest hole and re-secure the handwheel.



Adjusting the footrest

- The footrest is adjusted in height by releasing the handwheel as shown, the footplate is re-positioned and the handwheel is tightened to secure the new position
- The footrest is adjusted in angle by pressing the button as shown – on both side of the stroller. The desired angle is then set and the buttons released. This feature is also used when the seat is removed and folded.



Using the Tandem Buggy

- **GETTING INTO THE STROLLER**

- Make sure that the parking brake is on. Fold the footrest out of the way and recline the footrest rails as far back as possible. Either ask another helper to steady the stroller by holding the handle, or place the handle up against a wall to minimise the risk of tipping. Place the child in the stroller, fasten the harnesses, attach the occupant bar, and adjust the footrest.
- *The child should always feel comfortable and secure in the stroller.*

- **GETTING OUT OF THE STROLLER**

- Make sure that the parking brake is on. Reverse the procedure for getting into the stroller.

- **NEGOTIATING A KERB OR SINGLE STEP**

- To go down a kerb, approach the kerb until the front castors are near the edge. Grip the handle firmly and tip the stroller backwards to balance on the rear wheels. Lower the castors over the kerb on the rear wheels, returning the front castors to the ground when you have reached the lower level.
- To go up a kerb, reverse the procedure. Do not approach the kerb with the front castors. This may seem easier, but puts more strain on the stroller and also on you.
- *Never tip a stroller forwards at a kerb.*
- *Always make use of ramps and lifts if they are available.*
- *Do not attempt to mount a flight of stairs or steps.*

OCCUPIED TRANSPORT OF A TANDEM BUGGY IN A SUITABLY MODIFIED VEHICLE

- SECURING THE BUGGY TO THE TRACK SYSTEM IN THE VEHICLE FLOOR
- The TANDEM BUGGY must be secured to the vehicle floor using a 4-point webbing restraint system.
- **THE BUGGY MUST BE IN A FORWARD FACING POSITION WITH THE REAR SIBLING SEAT REMOVED.**
- Wheel the buggy into position and apply the brakes.
- The front of the buggy is secured first. Attach the front restraints to the track system making sure that they are locked in position. Pass the webbing around the tie-down point, which is clearly marked on the buggy chassis. Always attach to the vertical section of the frame – the strongest part of the construction.
- Now that the front of the buggy has been secured, carefully move the buggy backwards to take up the slack in the front restraints, and re-apply the brakes.
- The rear restraints are attached to the clearly marked attachment point. When the restraint is also attached to the track system in the vehicle floor, use the ratchet to pull the restraint tight. Make sure that you have tensioned the restraint on both sides of the buggy, and check that the buggy feels secure.
- THE RESTRAINTS SHOULD BE ATTACHED AT AN ANGLE OF NO MORE THAN 45 DEGREES TO THE VEHICLE FLOOR. IF YOU ARE IN ANY DOUBT, CONTACT THE RESTRAINT MANUFACTURER FOR SPECIFIC INSTRUCTION RELATING TO THEIR SYSTEM, OR YOUR DEALER FOR QUERIES RELATING TO THE BUGGY.



SECURING THE PASSENGER RESTRAINT AROUND THE OCCUPANT AND ATTACHING TO THE TRACK SYSTEM IN THE VEHICLE FLOOR.

- The following points relate to the safety of the occupant:
- **Check that occupant weighs no more than 35kg.**
- **Make sure that the seat base is at 90 degrees to the seat back. The seat back should be as upright as possible.**
- **No additional seating cushions should be used in transportation.**

The buggy occupant is protected by a 3-point passenger restraint, described by the restraint manufacturer as a SPLIT REEL DOUBLE INERTIA.

- The double inertia reel is attached to the track system on the track closest to the vehicle wall on which the upper torso anchorage is mounted. The stalk with the buckle on is placed opposite the double inertia reel on the other track. These should be positioned about 350mm/14" behind the buggy backrest to ensure a steep angle of application of the lap belt. The lap belt is reeled out from the front of the double inertia reel, fed between the buggy seat side and the backrest in order to maintain a steep angle over the pelvis. The belt must fit snugly over the pelvis and between the seat side and the backrest on the other side of the seat unit. The clip then attaches to the stalk, which is fixed to the track in the vehicle floor. The lap belt should be adjusted to be close fitting to avoid abdominal intrusion on impact.
- The upper torso restraint is reeled out of the rear of the double inertia unit. The leading clip is passed over the occupant's shoulder, and attached to the buckle on the lap belt. This will be found just above the lap belt attachment to the stalk. The remaining clip is then attached to the restraint anchorage on the vehicle wall. This should be adjusted to be at, or just above, shoulder level. The upper torso restraint should be adjusted to be close fitting, and comfortable for the occupant.



KEY POINTS RELATING TO TRANSPORTATION

- The buggy must face forwards in the vehicle – in the direction of travel.
- The buggy must be restrained with a 4-point tie-down. No other restraint system must be used.
- The occupant must be restrained by a 3-point passenger restraint.
- The passenger restraint *must* be anchored at, or above, shoulder level.
- The lap section of the passenger restraint should fit snugly over the pelvis of the occupant, at a steep angle of application to avoid abdominal intrusion.
- Check with the vehicle converter that the vehicle floor and side wall structures have the appropriate strength required by the restraint manufacturers for the installation of the restraint systems.
- Check that the vehicle converter has fitted the restraint system in conjunction with the restraint manufacturer. All manufacturer's recommendations and guidance notes should be followed.
- Check that all vehicle operators are suitably trained in the use, care and maintenance of the restraints.

CARE AND MAINTENANCE

- Your stroller has been designed and produced to a high standard. With correct use and maintenance it will give you many years of trouble-free service.
- Please read the following instructions.
- Regularly inspect the frame catch and the attached safety clip, the occupant bar fitting, locking devices, catches, the footrest and all fixings on the frame to ensure that they are secure and in good working order
- Brakes, wheels and tyres are subject to wear. If they need to be replaced, the work must be done by a qualified rehabilitation engineer.
- We recommend that the stroller is checked once a year by a qualified rehabilitation engineer.
- Any damage or repairs should be assessed by a qualified rehabilitation engineer, or should be returned to your dealer.
- Only genuine Patron Bohemia parts should be used when your stroller is repaired.
- Regular lubrication of moving parts will extend the life of your stroller and will make folding and unfolding easier. Use a spray lubricant and wipe off any excess spray. Any overspray will encourage a build up of dirt.
- Make sure that you are familiar with the folding method for your stroller. It should always fold easily, so do not force the mechanism if it does not. Stop and read this user guide again.
- Wheels and other plastic or metal parts may be sponged clean with warm water and a mild detergent.
- The fabric covers are removable, and should be washed in water no hotter than 40 degrees centigrade.
- To remove the backrest cover, remove any postural accessories and harnesses, unzip the top section and carefully remove the cover from the backrest. Make sure that you are able to replace the postural accessories in the right place.
- The seat cover is closely fitted over the seat sides, and is attached to the seat base with Velcro.
- Never clean the stroller with abrasive cleaners, ammonia based, bleach based or spirit based cleaners.
- After exposure to damp conditions, dry off your stroller with a soft cloth, and leave unfolded in a warm, dry environment.
- Store your stroller in a dry, safe place.



**For more information call Activate on
01722 340600
Or fax 01722 414884
Email: sales@activateforkids.com**