Eurovema

Creates mobility and ergonomics.

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INTRODUCTION

Thank you for choosing the ITO pushchair from Alvema. The pushchair is designed for disabled children to be used both indoor and outdoor. Please study the manual carefully and follow all safety instructions. This will ensure that you get the best use from your new chair.

CONTACT

If you have questions or need help with this product, please contact the local agent. To get in touch with the manufacturer see info below:

Eurovema AB
Baldersvägen 38
332 35 GISLAVED
SWEDEN
Tel:0371-390100
Fax:0371-189 82
E-post: info@eurovema.se
www.eurovema.se

SAFETY

- Read the manual carefully before use.
- Please note! The handle should be folded when lifting the pushchair or using on the stairs. Gripping the frame in front of the joint will give better control of the child and the pushchair and therefore be safer (20).
- Do not stand on the footplate. It should only be used for supporting the feet while sitting in the pushchair.

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SAFETY

- Do not leave children unattended in the push-chair.
- A bag attached to the frame adversely affects stability and may cause the chair to tip over.
- Never use defective equipment!
Do not attempt repairs yourself.
- For safety reasons, only an approved repairer or your supplier must carry out repairs. Only original spare parts from Eurovema must be used. Failure to observe this requirement will invalidate your guarantee.
- The harness included with the ITO is for positioning purpose only and should not be used as a dynamic restraint belt..

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UNPACKING THE ITO

- Remove the pushchair from the box (1).
- Unfold the pushchair with one action (2). Check the folding guards and engage them if necessary.
- Adjust the angle of the push-handle by pulling the capsules backwards. Set the angle and release (3).
- Place the cushion on the backrest, and attach the loops around the knobs on the back of the seat (4). Fix the seat cushion to the Velcro on the seat (5).
- Cover the pelvic supports with the pads and lock the Velcro (6).
- Fasten the lap belt with the straps around the bars on the side of the back using the snap fasteners (7).

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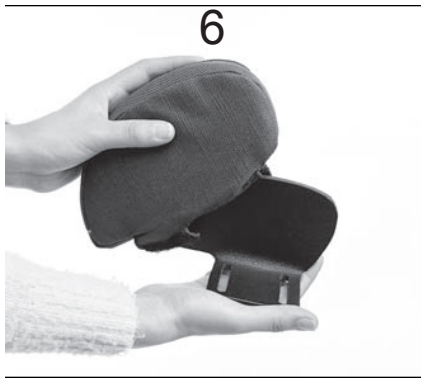
7

FOLDING THE ITO

- Lift the folding guards (8) and pull the capsules upwards (9).
- Push the back forwards (10). To minimize the volume of the folded pushchair, the back should be in the most upright position.
- If the back extender is used, the footplate needs to be folded, otherwise this is not necessary.
- Adjust the angle of the handle by pulling the capsules backwards (11).

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ADJUSTMENTS

Footplate adjustment

- Adjust the height of the footplate using a spanner/wrench (12). Release the nuts, set height and re-tighten (13).
- The mesh can be removed for washing by releasing the Velcro on the back of the footplate. Pull the straps out of the slots.

Width adjustments

- The seat width is adjusted by loosening the two outer screws under the seat (14). Adjust the pelvic supports to the desired position and angle and retighten the screws (15).

Depth adjustments

- The seat depth is adjusted by loosening the two screws on the tubes under the seat (16). Pull the seat to the desired length and retighten the screws (17).

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ADJUSTMENTS

Push-Handle adjustment

- To adjust the handle, pull the capsules towards your body (18). Set the angle and release the capsules (19).

Please note! The handle should be folded when lifting the pushchair or using on the stairs. Gripping the frame in front of the joint will give better control of the child and the pushchair and therefore be safer (20).

Tilt adjustment

- The tilt of the seat can be adjusted by pulling the bar upwards (21). Tilt the seat to the desired angle and release the bar (22).

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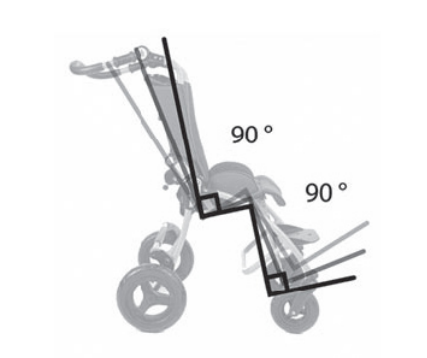
ACCESSORIES

Vest Harness, x-harness, h-harness

- Fix the plate with screws through the provided holes at desired height. Tighten with a nut on the back (23).
- Put the straps through the slots in the plate and fasten the snaps (24).
- Carefully cut open the back padding at the desired height and put the straps through the holes.
- Fasten the snaps around the lower bars in the back (25).

X-harness; Place the child in the pushchair and close the cam-lock buckles. Adjust the straps by pulling on them (26).

Vest harness; Place the child in the pushchair and close the zipper and the snap fastener. Adjust the straps by pulling on them (27).

H-harness; Place the child in the pushchair and close the cam-lock buckle. Adjust the straps by pulling on them (28).

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ACCESSORIES

Pelvic harness

- Attach the loose Velcro between the tubes under the seat (29).
- Place the pelvic harness on the seat and attach the rear straps to the Velcro underneath the seat.
- Attach the adjustable straps to the same bar as the lap belt (25).
- Adjust the straps over the thighs towards the back when the child is seated (30).

Lateral trunk supports

- Fix the plate with screws through the provided holes at desired height (31). There are several alternatives. Tighten with a nut on the back.
- Put the trunk support through the slots on the cushion with the elastic between the two attachment arms (32).
- Fit the attachment arms in the round holes by squeezing the support. Pull the support to the side and release the pressure in the desired position. For an alternative height it is possible to change sides on the supports.

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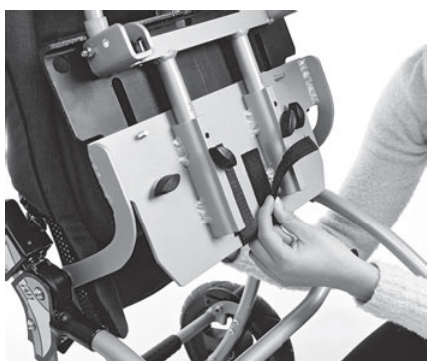
ACCESSORIES

Back extender

- Place the extender piece in the tubes on the back piece of the pushchair (33).
- Attach the loops around the knobs on the back of the seat (34).

Headrest

- Put the cords through the slots in the cushion and attach the loops around the knobs on the back of the seat according to the pictures, without the back extender (35), with the back extender (36).
- Adjust the height with the cord locks under the headrest (37).

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ACCESSORIES

Abductor

- Attach the fixing equipment to the front of the seat with the supplied screw and nut (38).
- Cut the padding open in one of the provided places.
- Pull out the knob and fit the abductor (39).

Grip rail

- Loosen the screws in the front of the seat's sides (40).
- Place the attachment appliance by the hole and use the screw and the nut to tighten it (41).
- Remove the plastic caps and slide the grip rail down into the holes (42) while pressing in the two knobs on the sides. Also press the two knobs when releasing the grip rail.
- The grip rail can be released on one side to be turned up and give better access to the chair (43)..

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ACCESSORIES

Knee cushion

- Attach the knee cushion to the Velcro on the seat. The larger side should be placed in the direction of movement.
- Place the cushion on top of the knee cushion (44).

Lumbar air cushion

- Attach the lumbar air cushion to the back padding at a suitable height (45).
- Pull the pump out between the back and the seat.
- Inflate to a comfortable size (46).

Please note! Only a very small amount of air is needed, usually just one or two pumps are required. Please try the equipment on your own back to realise what might be comfortable.

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ACCESSORIES

Sun-canopy

- If the extension piece is used, open the zipper 2 cms to get access to the tubes in the back.
- Insert the sun-canopy into the two tubes on the back of the seat or in the tubes in the extension piece (47).
- Attach the loops around the upper knobs on the back of the seat, if the extension piece is used, otherwise use the second knobs (48).

Sun-canopy cover

- Pull the opening in the cover over the handle.
- Put the elastic over the front piece of the sun-canopy.
- Put the straps around the frame and fasten the snaps (49).

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ACCESSORIES

Footstraps

• Fix the foot to the footplate by running the strap through the four holes around the foot. Attach the ends across the foot and adjust the length until the foot is firmly in place. The straps can be fixed x style or parallel across the foot (50).

Strap covers

• Put the cover over the strap or the buckle where it is needed (51).

Basket

• Close the zippers around the two bars under the pushchair (52).

• Put the straps around the frame and fasten the snaps (53).

Wind shield

• The wind shield can be attached with or without the child in the seat.

• Put the footrest in the bag (54).

• Attach the loops around the knobs on the back of the seat (55).

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ACCESSORIES

Thermal bag

- Attach the loose Velcro between the tubes under the seat (56).
- Place the bag on the seat and attach the rear straps to the Velcro underneath the seat.
- The entire front piece can be removed with the zipper to give better access to the seat.
- Adjust the cord lock at the top.

Rain cape

- Pull the hood over the child's head (57).
- Open the Velcro on the back of the cover and attach around the handle (58).
- Put the elastic around the back of the push-chair (59). (If the sunroof is used put the elastic around the child's back.)
- Put the lower elastic around the footrest.

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CHANGING WHEELS

From castor wheels to fixed wheels

- Loosen the screw on the wheel axis and remove the castor wheel (60).
- Loosen the nut under the fork with a wrench.
- Loosen the screw on the frame and remove the fork (61).
- Put the fixed wheel in place and tighten the screw on the frame (62).
- Tighten the screw on the middle bar.

From fixed wheels to castor wheels

- Loosen the screws on the middle bar and on the frame (63). Remove the wheel.
- Put the fork in place and tighten the screw on the frame (62).
- Tighten the screw under the fork with a wrench (61).
- Mount the castor wheel and tighten the screw in the axis (60). Use locktite on the screws.

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63

ATTACHING THE HAND BRAKE

- Remove the capsule with a tool (64).
- Remove the wheel with a tool (65).
- Mount the brake on the wheel axle and attach it to the frame with a screw and a nut (66).
- Remount the wheel.
- Place the wire behind the back cushion (67) and attach the cushion to the back again using the knobs on the back (68).
- Attach the braking handle to the push-handle with a tool (69). Tighten the screws.

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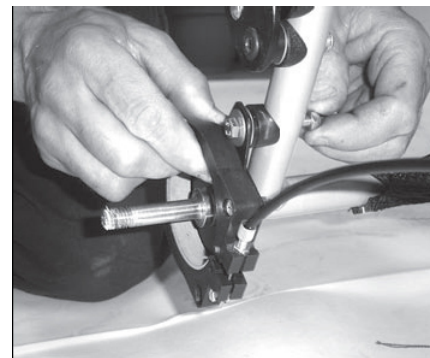
69

MAINTENANCE AND CARE

The cushion can be washed at 40°C, spun and drip-dried. Do not use fabric softener or detergents with brightening agents. Iron after washing for best maintenance of the finish.

Do not dry clean!

Store the pushchair in a dry place out of bright sunlight.

The frame should be cleaned with a cloth and car shampoo/polish.

A thorough check up of the equipment should be performed by authorized personal twice a year. Immediately contact your reseller if a fault should appear.

english

TRANSPORT

Please, follow these instructions to avoid injury of the child or/and damage of the Pushchair during transit in motor vehicles

The ito pushchair is designed to meet the requirements of ANSI/RESNA WC/Vol. 1-Section 19 regarding transportation of a wheelchair in a vehicle.

The ITO is capable of transporting users who are the following sizes:

- Small ITO - Maximum Height = 47" (120cm)
Maximum Weight = 55 lbs. (25kg)
- Large ITO - Maximum Height = 57" (145cm)
Maximum Weight = 88 lbs. (40kg)

The ITO has been dynamically crash tested for use with an approved four point wheelchair tie-down system, such as the "Q'Straint (with lap and shoulder belt.) The ITO showed minimum lateral displacement (2mm) when tested under ANSI/RESNA WC-19 Lateral Stability test using a standard tie-down system. The illustration opposite indicates the four anchor locations for securing the wheelchair tie-down system to the ITO pushchair. Refer to the manufacturer's tie-down instructions for further information.

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TRANSPORT

The harness included with the ITO is for positioning purpose only and should not be used as a dynamic restraint belt. Please follow the wheelchair tie-down restraint manufacturer's instructions for securing the occupant restraint belt.

The ITO is rated with a "B" (Good) with respect to the use and fit of vehicle anchored belt restraints under ANSI/RESNA WC-19.

Keep in mind that the tie-down manufacturer's pelvic belt should be worn low across the front of the pelvis so that the angle of the belt is 45 to 75 degrees from horizontal. Belt restraints should not be held away from the body by wheelchair components or parts and the upper torso belt should fit over the shoulders. Belt webbing should not be worn twisted in a manner that significantly reduces the area of contact of the belt with the occupant

The ITO has four anchor locations.

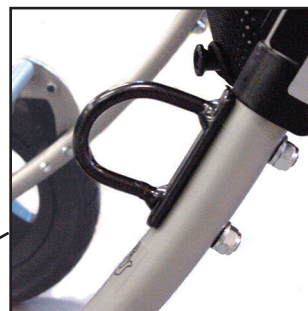
Two on the back legs (A+B), two on the front legs (C+D). The anchor points are "D" rings bolted to the chassis .

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TRANSPORT

Warnings:

The ITO pushchair should be used only for forward facing seating in motor vehicles.

The ITO should be used only as indicated in the manufacturer's instructions.

The ITO has been dynamically tested in a forward facing position with the use of both pelvic and shoulder belts. Both pelvic and upper torso (shoulder) belts should be used to reduce the possibility of head and chest impacts with vehicle components.

The ITO harnesses can be used for postural support, but cannot be relied on for occupant restraint. A dynamically tested vehicle anchored, lap and shoulder belt restraint should be used to restrain the occupant. A clear zone of 65cm (25") should exist in front of the child to prevent the child's head from contacting anything in a crash.

Auxiliary equipment attached to the ITO should be effectively secured, or removed and secured in the vehicle during transit, to prevent it from breaking free and causing injury to vehicle occupants in a crash.

Unauthorized alterations should not be made to the ITO since they may affect its performance in a crash.

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Material:

During manufacturing only non-toxic paint and powder coating is used. All of the upholstery is non-flammable according to standard ISO EN 1021-1 and 1021-2. Steel and aluminium are used in all tubes. The cushions are washable and tagged with simple cleaning instructions. The material has also been tested for biocompatibility according to standard ISO 10993-5.

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Test:

It is crash tested and approved according to standard ANSI / RESNA WC / Vol. 1 and ISO / FDIS 7176 and also approved by TÜV according to standard DIN EN 12182 and EN 1888.

Test:

It is crash tested and approved according to standard ANSI / RESNA WC / Vol. 1 and ISO / FDIS 7176 and is also approved by TÜV according to standard DIN EN 12182 and EN 1888.

*Euro*vema
Creates mobility and ergonomics